

## Dräger DrugTest® 5000 Test Kit

The Dräger DrugTest® 5000 system is a fast, accurate means of testing oral fluid samples for drugs of abuse, such as amphetamines, designer amphetamines, opiates, cocaine and metabolites, benzodiazepines and cannabinoids, as well as precise diagnostic evaluation and data management.



The Dräger DrugTest® 5000 system is comprised of two main components: the Dräger DrugTest® 5000 Test Kits and the Dräger DrugTest® 5000 Analyzer.

Ergonomically designed for easy handling and ready for immediate use, this innovative test kit consists of one major part, a test cassette complete with oral fluid collector. Simple to operate with either the left or right hand, with or without gloves, it avoids the embarrassment inherent with other drug detection collection methods. The Dräger DrugTest® 5000 Test Kit oral fluid collector is made from a rigid, porous material with capillary properties. This non-invasive test is also hygienic to use because it inhibits contact with the sample and its components.

Minimizing the risk of operator error, the test is carried out in five straightforward steps. First, the protective cap is removed from the oral test collector and the test cassette is handed to the donor. The donor then places the collector inside his or her mouth and, by using a sideways motion between the cheek and gum, takes an oral fluid sample.

As soon as sufficient fluid has been gathered, the built-in indicator will turn blue and the test cassette can be handed back to the operator. The operator, who will have observed the whole process, then places the test cassette and cartridge into the analyzer and awaits the result which will be clearly displayed for each drug class within a few minutes.

Individual samples can be easily identified by simply writing the name and date on the test cassette housing. Test cassette expiration dates are stored within each individual cassette's bar code and is displayed when inserted into the analyzer thus eliminating the possible use of an expired cassette. In addition, each bar code contains information for that lot of cassettes and calibration of the analyzer occurs with each inserted cassette. If required, the Dräger DrugTest® 5000 can be used in conjunction with the Dräger DCD 5000 oral fluid collection device to provide second sampling, storage and transportation for confirmation laboratory analysis purposes.



The portable Dräger DrugTest® 5000 Test Kit: easy, fast, reliable and safe drug detection

## ORDER INFORMATION

Each Dräger DrugTest® 5000 Test Kit consists of one pouched sample collection and test cassette and one cartridge.

Item	Order No.
Dräger DrugTest® 5000 Training Test Kit (20 per box)	83 19 970
Dräger DrugTest® 5000 Test Kit 6-drug panel (USA version-20 per box)	83 19 965

### Accessories


Dräger DCD 5000 Device for collecting oral fluid samples (20 per box)	83 19 910
---	-----------

## DRUG ANALYTICAL CUT-OFF

Dräger DrugTest® 5000 Test Kit  
6 Panel Cut-Off Levels


AMP	50 ng/ml
D-amphetamine	
MAMP	35 ng/ml (MDMA 100)
D-methamphetamine	
COC	20 ng/ml
Cocaine	
OPI	20 ng/ml
Morphine	
CAN	5 ng/ml
D9-tetrahydrocannabinol	
BENZO	15 ng/ml
Diazepam	

## DRÄGER DRUGTEST® 5000 TEST CARRIED OUT IN FIVE STRAIGHTFORWARD STEPS:

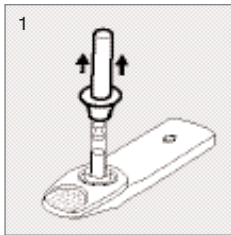


ST-319-2008

Remove protective cap .

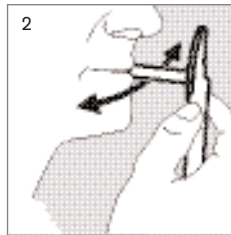


ST-320-2008

Place the collector inside mouth.

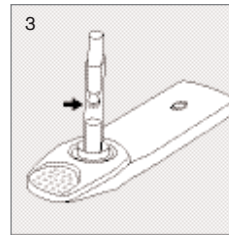


ST-321-2008

The built-in indicator will turn blue.

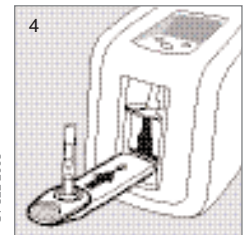

ST-322-2008

Test cassette and cartridge will be placed into the analyzer.

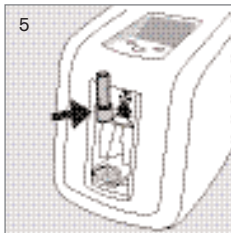

ST-323-2008

The result will be clearly displayed.

### Storage and Stability

The Test-Kit must be stored at a temperature between 4 and 30 °C (39 to 86 °F) in its original foil pouch. Do not use the test kit if the foil pouch, in which it is contained is damaged, for example with a hole or tear. The test kit must be used immediately after opening and must never be used after the expiration date, which is clearly printed on the foil pouch and will be displayed when placed into the analyzer.

[www.draeger.com](http://www.draeger.com)

#### USA

Dräger Safety Diagnostics, Inc.  
4040 West Royal Lane, Suite 136  
Irving, TX 75063

#### USA

Tel +1 972 929-1100  
Fax +1 972 929-1105

[info@dstdi.biz](mailto:info@dstdi.biz)

#### CANADA

Dräger Safety Canada Ltd.  
7555 Danbro Crescent  
Mississauga, ON L5N 6P9  
Canada

Tel +1 905 821-8988  
Fax +1 905 821-2565

[sales.canada@draeger.com](mailto:sales.canada@draeger.com)

#### MEXICO

Dräger Safety S.A. de C.V.  
Av. Pe-uelas No. 5  
Bodega No. 37  
Fraccionamiento Industrial  
San Pedrito  
Queretaro, Qro C.P. 76148  
Mexico

Tel +1 52 442 246 1113  
Fax +1 52 442 246 1114  
[ventas.mexico@draeger.com](mailto:ventas.mexico@draeger.com)

#### GERMANY

Dräger Safety AG & Co. KGaA  
Revalstrasse 1  
23560 Lübeck  
Germany

Customer Service  
Tel +49 451 882-0  
Fax +49 451 882-2080